



Osso VR Hardware & Use Training

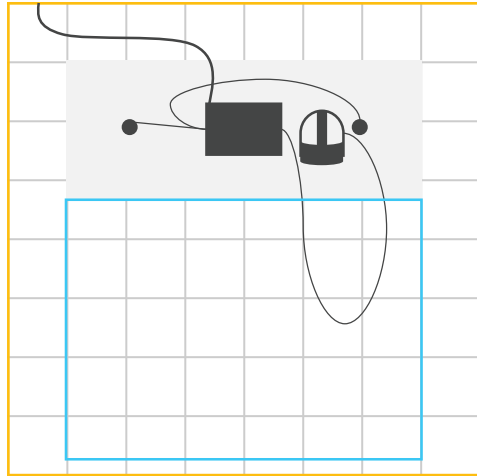


1

Set up
hardware
& area

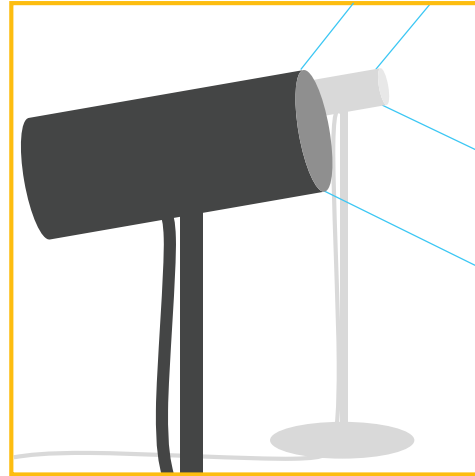

2


Calibrate
sensor
cameras

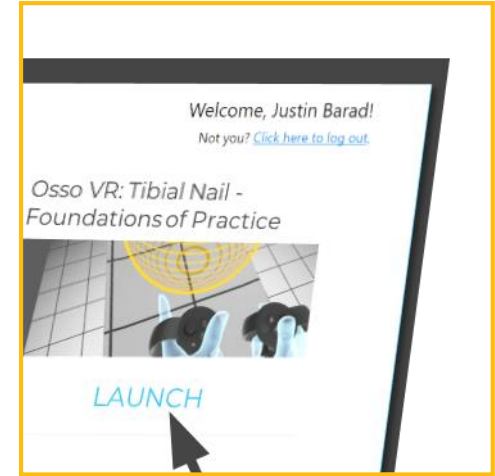

3

Launch
training
scenarios

Osso VR Hardware & Use Training

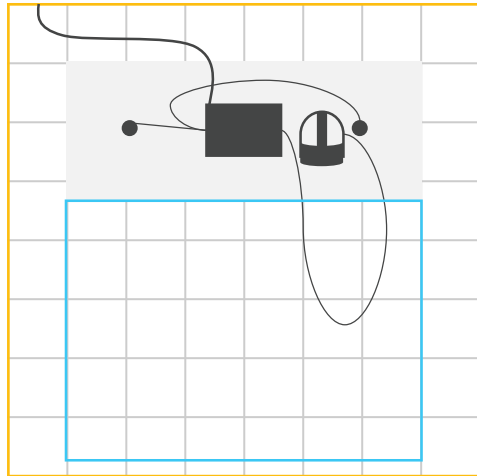

1

Set up hardware & area


- ✓ 6 X 6 feet of open space
- ✓ Minimum of 4 ft wide table or surface area (standard 6 ft table works great)
- ✓ Sensor cameras placed ~1 foot from edge of table and ~1 foot from side of laptop with shiny side in front.

- ✓ Point sensors directly forward and slightly upward (not inward)

TIPS:

1. Plug in laptop for best training experience.
2. External displays are great for multiple person sessions.

Osso VR Hardware & Use Training

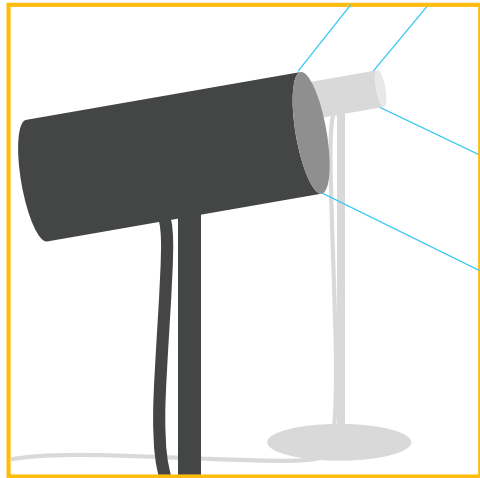

2

Calibrate
sensor
cameras


- ✓ Turn on laptop and log in with provided password.*
- ✓ Open Oculus Rift application from task bar.
- ✓ Go to **Devices** and **Configure Rift** and the **Sensor Setup** option
- ✓ Follow prompts
- ✓ Go through steps within headset.

TIPS:

1. Do not enable Guardian system. Skip this step during setup.
2. If table height is the same as the last use, this step might be skipped. See [Shortcut document](#)

**For your unit password, contact your organization's Osso VR lead.*

Osso VR Hardware & Use Training

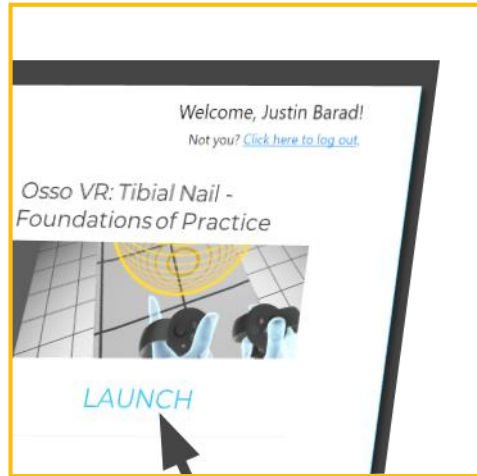

3 Launch training scenarios

- ✓ Ensure laptop is connected to internet.
- ✓ Open Osso VR app from task bar.
- ✓ Sign-in (or select sign-up option)
- ✓ Log user by collecting email address
- ✓ For self directed training or demos, select "Run as Supervisor"
- ✓ Launch desired training scenario from tile menu.

TIPS:

1. Ensure every user signs in after initial login.